CORAZOR® Biopsie / *Biopsy*

Schlüssel zur sicheren Diagnose
The key to safe diagnosis

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CORAZOR® UROMED BIOPSIE-SYSTEM – Konzi

UROMED BIOPSY SYSTEM – Designed for highest demands.

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Smooth guidance by special polishing of the »CORAZOR®« Puncture Biopsy Cannula – even when cannula is bended.

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In comparison: the cross-section of other customary puncture biopsy cannulas.

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Excellent quality of the biopsied tissue because of the sharp cut.

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The »CORAZOR®« Biopsy Device is worldwide the first one offering semi-automatic and rapid handling. It can be used one-handed and features a high spring tension. The 3-point-fixation of the cannula guarantees a safe guidance. The preparation and the biopsy itself respecting sterility and highest possible safety for the staff are ensured by a special contamination protection.

The »CORAZOR®« Puncture Biopsy Cannulas and the new »CORAZOR®« Biopsy Device are perfectly synchronized to each other. The system permits an exceptionally precise and point-exact sampling of tissue – especially in connection with image-guided procedures. The achieved high detection rates increase the diagnostic certainty in cases of suspected carcinoma considerably.

Designed for highest demands.

- Precise guidance.
- Easy cocking mechanism.
- Highest penetration power.

Phenomenal results.

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INHALT

CONTENTS

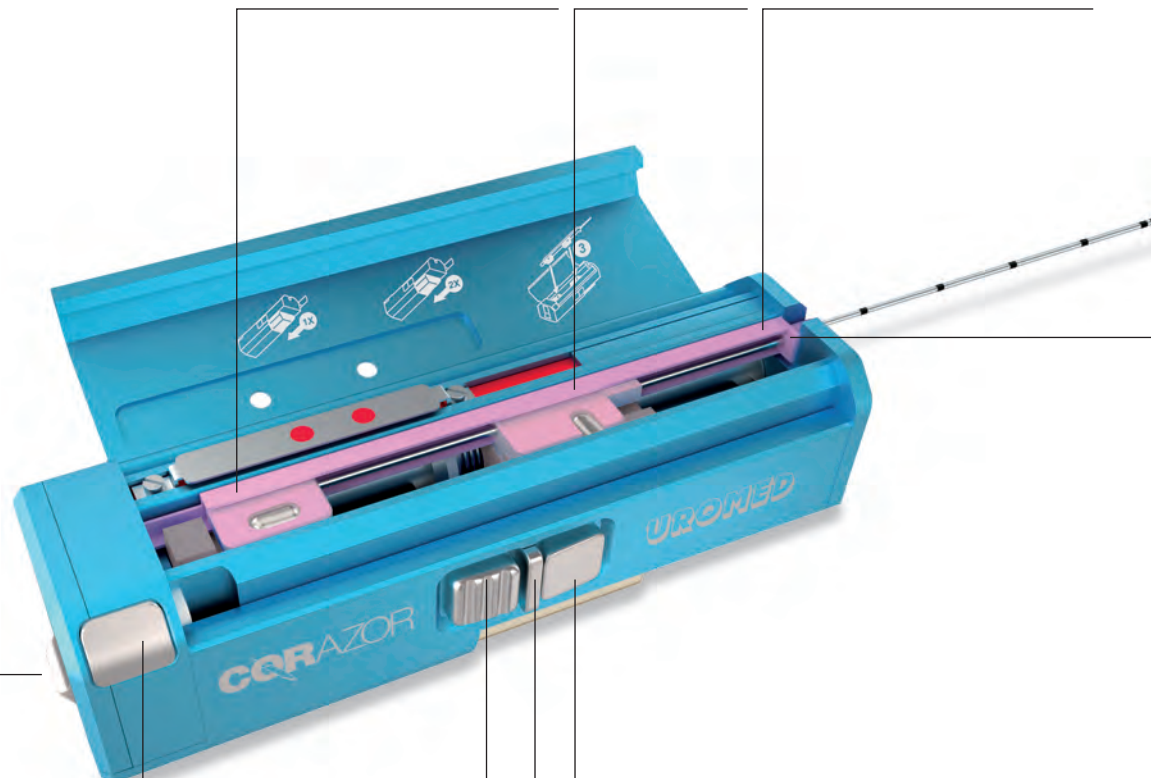
»CORAZOR®« Biopsie-System <i>»CORAZOR®« Biopsy System</i>	4
UROMED Stanzbiopsie-Kanülen <i>UROMED Biopsy Cannulas</i>	6
UROMED Biopsie-Schussgerät »CORAZOR®« <i>UROMED Biopsy Device »CORAZOR®«</i>	8
UROMED Kanüle zur Feinnadelbiopsie und Koaxial-Schleuse <i>UROMED Cannula for fine-needle biopsy and for percutaneous puncture</i>	10

CORAZOR® UROMED BIOPSIE-SYSTEM – zielg

UROMED BIOPSY SYSTEM – precise, accurate and safe.

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Optimized guidance of the cannula by 3-point-fixation.

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Direct opening of the cover by pushing the button.

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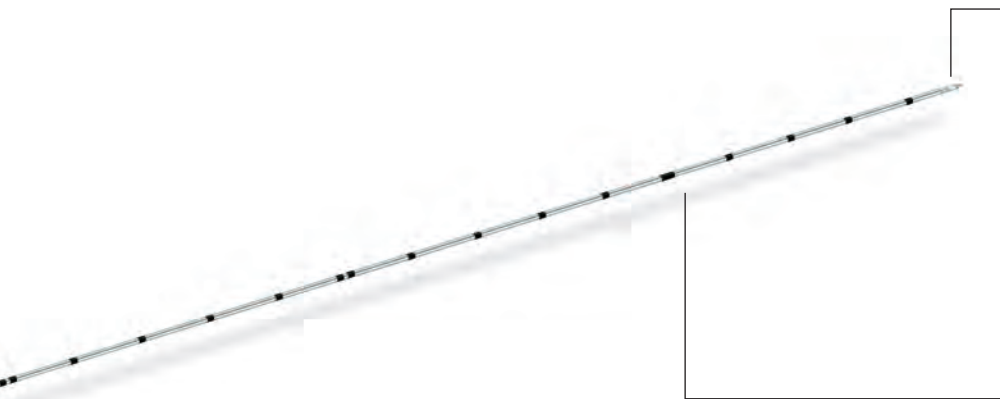
Different triggers for semi-automatic or automatic biopsy and special pressure point protection against accidental release.

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Excellent tissue results because of the special cut of the cannula.

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Smooth guidance by special polishing – even when the cannula is bended.

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Contamination protection by extended plastic attachment avoiding contact of the cannula with the device.

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The protection tube is firmly connected to the cannula which guarantees sterility and safety for the user.

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Right before puncture separate the protection tube from the cannula by turning the handle through 90°.

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Thereafter the protective tube can be pulled off easily.

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*Puncture Biopsy Cannulas
for reusable biopsy
instruments*

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Details with great effect:

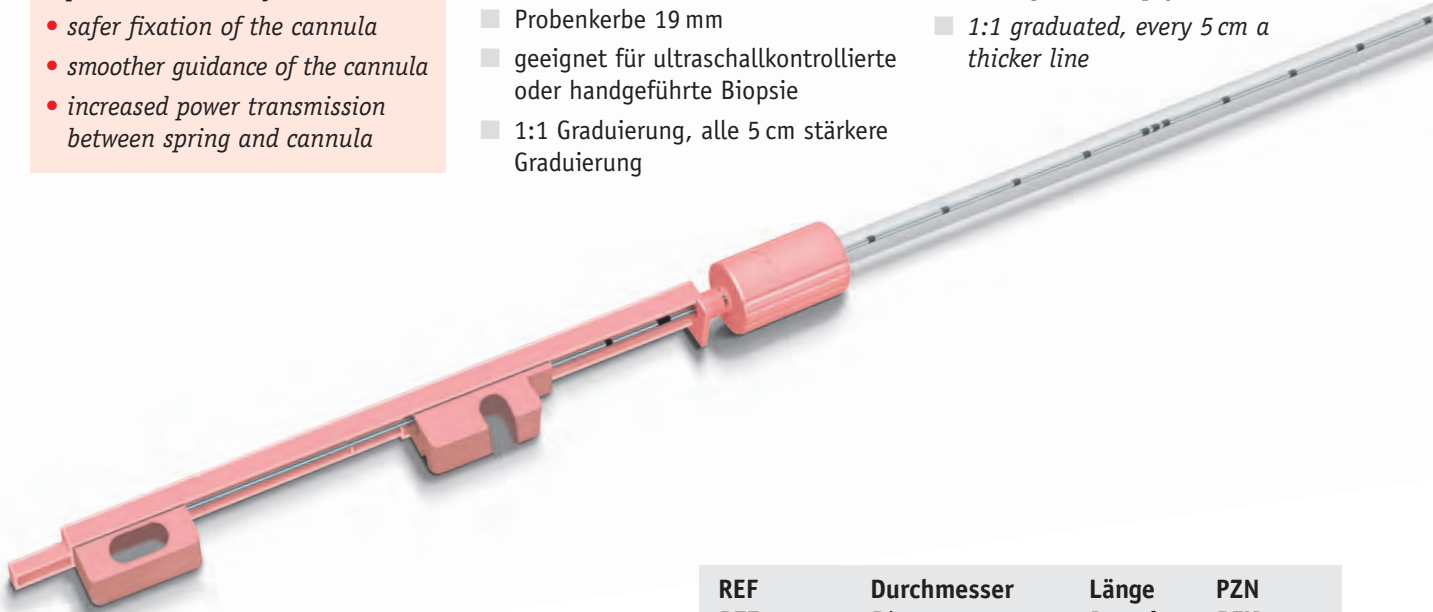
- precise insertion of the cannula
- safer fixation of the cannula
- smoother guidance of the cannula
- increased power transmission between spring and cannula

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*For biopsy and soft tissue
sampling*

- axial-rotation-cut for very high tissue results
- optimal guidance due to excellent 3-point-fixation of inserted cannula
- smooth guidance by special polishing – even when cannula is bended
- with protective tube for prevention of needlestick injuries and sterility – even with closed and cocked instrument
- contamination protection by extended plastic attachment which avoids the contact of the needle with the instrument
- easy insertion of the cannula
- luer connection for aspiration or application of local anaesthesia
- notch length 19 mm
- suitable for ultrasound or hand-guided biopsy
- 1:1 graduated, every 5 cm a thicker line

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Packing: sterile
Packing unit: 10 pieces
Original carton: 120 pieces

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Using of »CORAZOR®«
biopsy device in semi-
automatic

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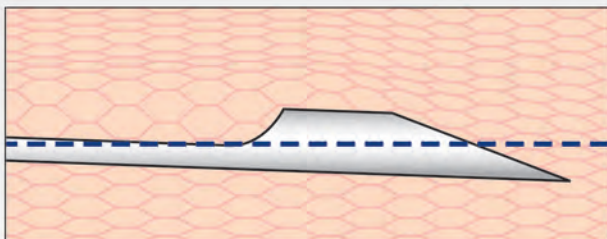
For biopsy and soft tissue
sampling

- more precise diagnostic of carcinoma due to high point-exact sampling of tissue because of using triangular trocar shape
- best possible tissue sampling due to triangular trocar shape and sample notch
- optimal guidance due to excellent 3-point-fixation of inserted cannula
- smooth guidance by special polishing – even when cannula is bended
- with protective tube for prevention of needlestick injuries and sterility – even with closed and cocked instrument
- contamination protection by extended plastic attachment which avoids the contact of the needle with the instrument
- easy insertion of the cannula
- notch length 19 mm
- suitable for ultrasound biopsy
- 1:1 graduated, every 5 cm a thicker line

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
Deviation of cannula when using conventional
beveled needles

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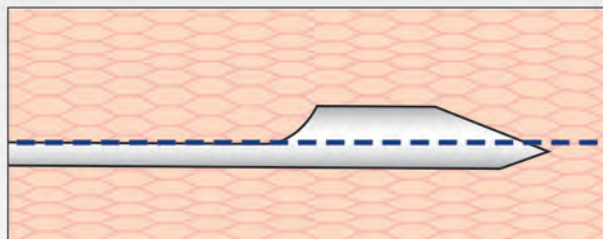More precise tissue sampling due to less deviation
based on use of triangular trocar-shaped needles

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Optimal line for sampling of prostate tissue exactly in the
target area

REF REF	Durchmesser Diameter	Länge Length	PZN PZN
6025.20	18 G / 1,20 mm	200 mm	10391697
6025.24	18 G / 1,20 mm	240 mm	10267603

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*UROMED Biopsy Device –
The Innovation in accurate
biopsy.*

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For biopsy and soft tissue sampling

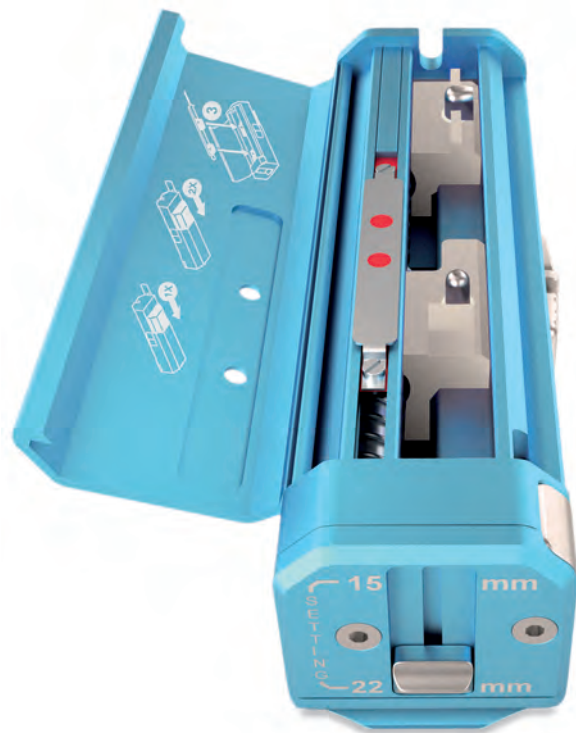
- *high spring tension for powerful puncture and optimized tissue samples*
- *triggers for automatic or semi-automatic sampling*
- *time extensive release is not necessary due to special pressure point protection*
- *excellent one-handed use*
- *easy cocking*
- *quiet application*
- *opening of the cover by pushing the button*
- *optional penetration depth 15 or 22mm*
- *status of the device is shown on the cocking indicator*
- *customised application with »CORAZOR®« Puncture Biopsy Cannulas*

REF anwendbar mit »CORAZOR®«-Stanzbiopsie-Kanülen
REF usable with »CORAZOR®« puncture cannulas

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
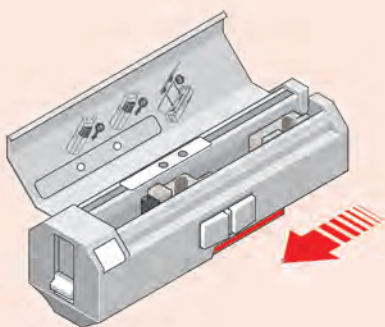
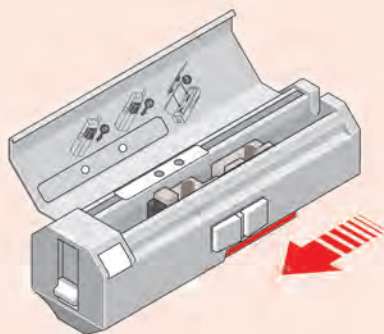
*Before (!) inserting the cannula the biopsy device has to be cocked **twice** up to the limit.*

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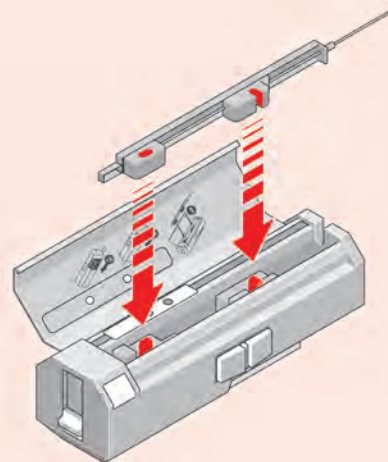After the second cocking the device is ready to insert the cannula.

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Insert cannula and close the cover. Safety release is not necessary – the puncture can be carried out with one hand.

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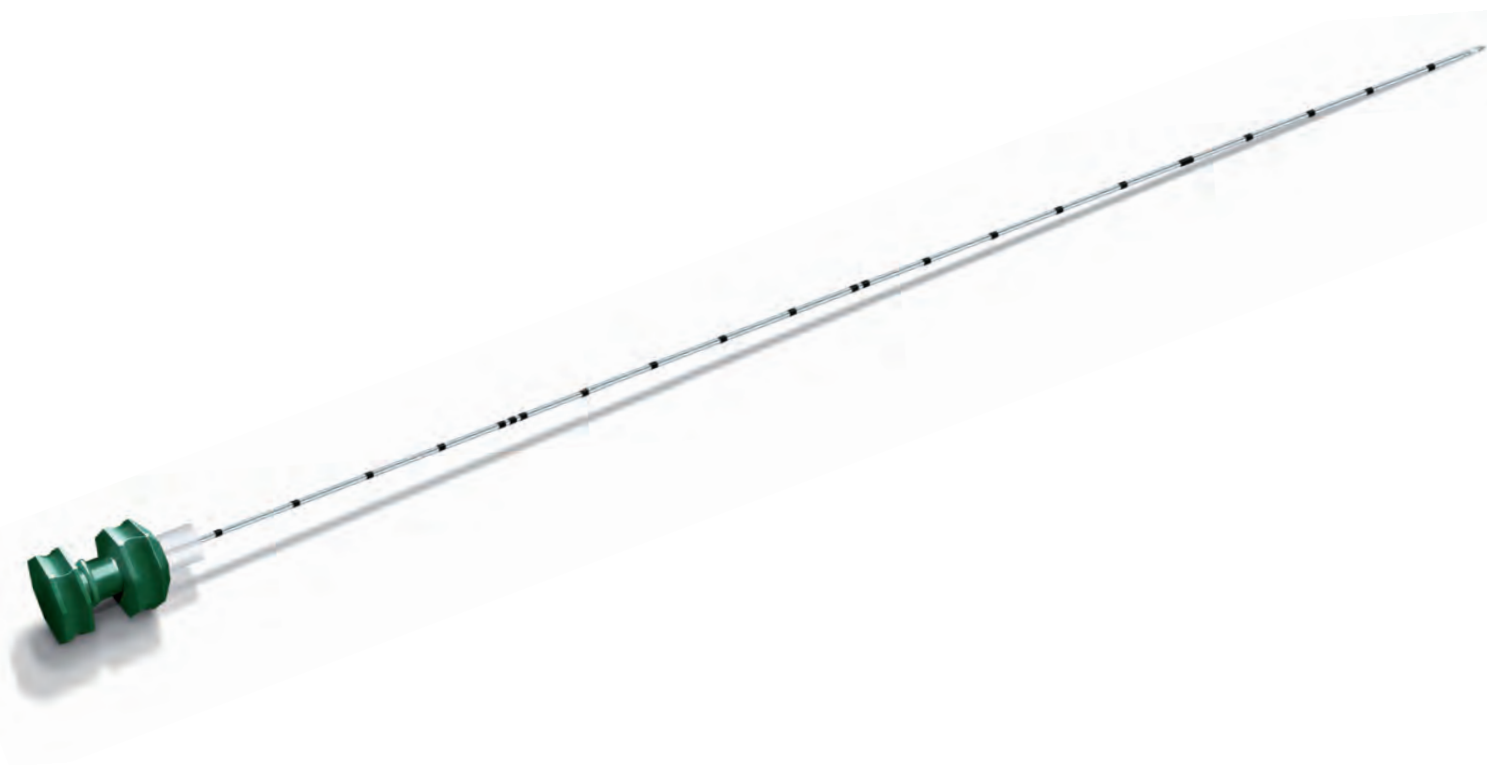
Cannula for fine-needle biopsy

- *high stability by Luer connection between outer cannula and inner mandrin*
- *inner mandrin with trocar-cut*
- *flexible depth regulator for controlling penetration depth*
- *1:1 graduated from the tip, every 5 cm a thicker line*

REF <i>REF</i>	Durchmesser <i>Diameter</i>	Länge <i>Length</i>	PZN <i>PZN</i>
6065.20	21 G / 0,80 mm	200 mm	09121136

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Cannula for percutaneous puncture

- high stability by Luer connection between outer cannula and inner mandrin
- inner mandrin with trocar-cut
- flexible depth regulator for controlling penetration depth
- 1:1 graduated from the tip, every 5 cm a thicker line
- usable with puncture cannula REF 6050.16, 18 G / 160 mm

REF <i>REF</i>	Durchmesser <i>Diameter</i>	Länge <i>Length</i>	PZN <i>PZN</i>
6056.13	16 G / 1,20 mm	119 mm	06438111

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*All indications correspond to our know-
ledge at the date of press.
Due to technical progress all products are
subject to change.*

05 / 2014

© UROMED Kurt Drews KG 2014

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UROMEDEX Export GmbH