


What is the AMT Introducer?

The AMT introducer is a device designed to stiffen the shaft of a Mini low profile balloon button (Mini Classic) making it easier to change.

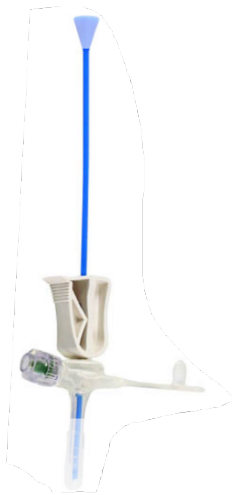

It may be useful to use the AMT Introducer with the Mini-Button if the stoma tract is:

- Long
- Tight
- Not straight Making button changes easier.

The AMT Introducer should only be used with an AMT Mini balloon button it is not designed for use with other types of gastrostomy device.

The AMT Introducer comes in two parts a clamp and a soft plastic stiffening stylet.

1. Insert the rounded tip of the clamp into the new Mini-Button feeding port.
2. Insert the stiffener through the clamp until it reaches to just before the end of the Mini-Button shaft.
3. Lock the stiffener in place by squeezing the AMT Introducer Clamp shut. Verify the stiffener is locked in after clamping.
4. Insert the Mini-Button into the stoma as normal. It may be useful to hold the shaft of the Mini-Button close to the tip to initiate entry into the stoma.
5. Once the Mini-Button is fully inserted, unclamp the AMT Introducer Clamp and remove the stylet



IMPORTANT: To avoid damage to the Mini-Button's internal valve, always remove the stylet before the AMT Introducer Clamp.

6. Gently remove the AMT Introducer Clamp while holding the Mini-Button's external bolster against the patient to prevent inadvertent feeding device removal.
7. Inflate the retention balloon to the volume shown on the balloon inflation port.

The AMT Introducer is only designed to be used with Mini low profile balloon button (Mini Classic) It is not designed for use with other gastrostomy type buttons.



For a complete list of products, visit us at <https://www.kebomed.co.uk/products/amt-enteral-devices/>

01884 266377 | enteral.sales@kebomed.co.uk

KeBomed UK
Swaines Hill Manor
Estate Offices, Alton
GU34 4DP

KEBOMED