


MiniACE® Button

Low Profile Antegrade Continence Enema Button

LEGACY + ENFIT®


Replacing the MiniACE® Button*

To ensure a proper fit, it is crucial to measure the stoma length. This is not a substitute for the DFU/IFU (Directions for Use/Instructions for Use).

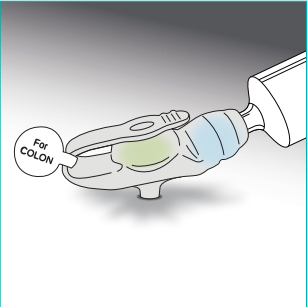


DEFLATE BALLOON

Deflate the existing device using a Luer syringe or Luer Lock syringe.

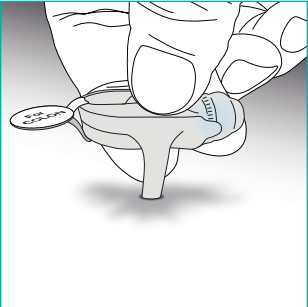


REMOVE DEVICE

Remove the device, gently pull up on the device with slow and steady pressure.

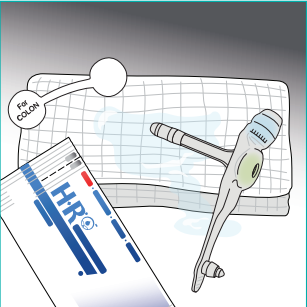


PREPARE KIT

Prepare the new device, apply a water soluble lubricant to the device.

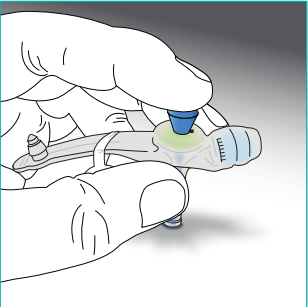

INSERT DEVICE

Insert the device into the stoma site, until it is flush against the skin. Remove the blue stiffener.

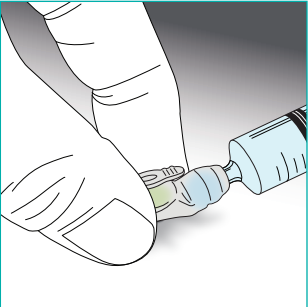


FILL BALLOON

Fill the balloon to the recommended fill volume, which is printed on the balloon fill valve.


Irrigation Set Attachment

The connection between the MiniACE® Button and its irrigation set is 10x stronger than the competition.¹

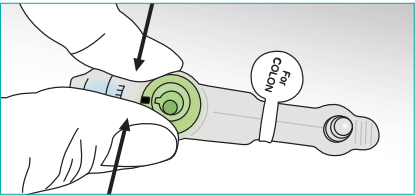


STABILIZE

Stabilize the device.

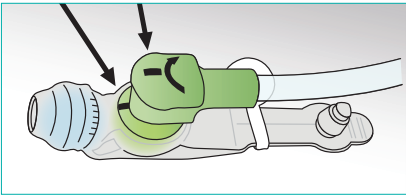


INSERT

Insert connector end.

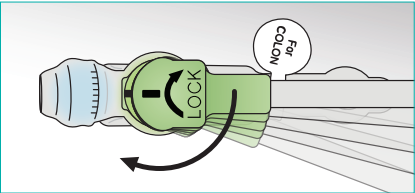

ROTATE

Rotate the connector 3/4 to the right.

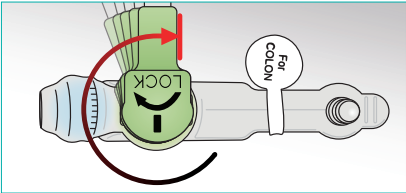

LOCK

Lock the connector into place, now the device is ready to irrigate.

Legacy + ENFit®

Available MiniACE® Enema Button Sizes: 10F, 12F, 14F



F - Tube Diameter

cm - Stoma Length

Example:


Size: 14F x 2.3 cm
Order#: ACE-1423

Cross Reference Guide*

MiniACE®				Chait Trapdoor™			
	10 FRENCH	12 French	14 French	10 FRENCH			
Length	ORDER #						
0.8 cm	ACE-1008	ACE-1208	ACE-1408	G11632 0-6 cm			
1.0 cm	ACE-1010	ACE-1210	ACE-1410				
1.2 cm	ACE-1012	ACE-1212	ACE-1412				
1.5 cm	ACE-1015	ACE-1215	ACE-1415				
1.7 cm	ACE-1017	ACE-1217	ACE-1417				
2.0 cm	ACE-1020	ACE-1220	ACE-1420				
2.3 cm	ACE-1023	ACE-1223	ACE-1423				
2.5 cm	ACE-1025	ACE-1225	ACE-1425				
2.7 cm	ACE-1027	ACE-1227	ACE-1427				
3.0 cm	ACE-1030	ACE-1230	ACE-1430		G36392 3-9 cm		
3.5 cm	ACE-1035	ACE-1235	ACE-1435				
4.0 cm	ACE-1040	ACE-1240	ACE-1440				
4.4 cm	ACE-1044	ACE-1244	ACE-1444				
5.0 cm	ACE-1050	ACE-1250	ACE-1450				
5.5 cm	ACE-1055	ACE-1255	ACE-1455				
6.0 cm	ACE-1060	ACE-1260	ACE-1460	G12685 6-14 cm			
6.5 cm	ACE-1065	ACE-1265	ACE-1465				
7.0 cm	ACE-1070	ACE-1270	ACE-1470				
8.0 cm	ACE-1080	ACE-1280	ACE-1480				
9.0 cm	ACE-1090	ACE-1290	ACE-1490				
10.0 cm	ACE-10100	ACE-12100	ACE-14100				

Kit Irrigation Sets

LEGACY KIT – 2-1211

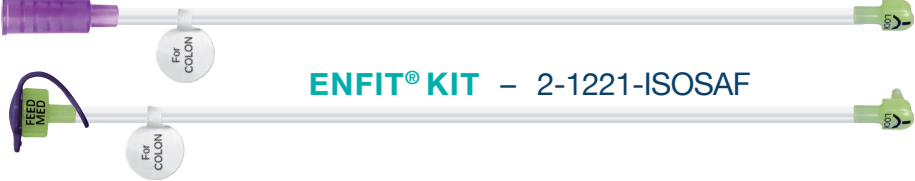

ENFIT® KIT – 2-1221-ISOSAF

Questions? Call Today:



For a complete list of products visit us at www.AppliedMedical.net
800 869 7382 | CS@AppliedMedical.net | ICS@AppliedMedical.net

